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## Contents

### Preface

- 7. Purchased parts package
- 8. Intended use
- 9. Warning notice
- 10. CE conformity declaration

### Operation

- 11. Assembly
- 12. Usage in combination with the Symphonie Aqua System
- 13. Guarantee

**This operating manual is designed to provide you with the safest and best possible handling. Please read the instructions completely and attentively before initial operation and carefully store them for future accessibility. We take no liability for any claims that result from inobservance of this operating manual.  
Product images may vary.**

## **7. Purchased parts package**

The Symphonie Aqua Lifter is already completely assembled upon delivery. You only need to remove it from the packaging and choose an appropriate location for it in your workshop.

1x Symphonie Aqua Lifter

## **8. Intended use**

Dear Symphonie Aqua User,

Thank you for choosing the Symphonie Aqua Lifter. This innovative lifter was especially developed for use in combination with the Symphonie Aqua VC TF System. The quality products from the Romedis GmbH undergo extensive quality assurance and documentation, as well as close-meshed tests in order to ensure flawless production.

Operate the lifter only as instructed in the operating instructions and only in combination with the Symphonie Aqua VC systems.

Any hazard occurring due to improper usage is the responsibility of the operator or user, and not the manufacturer.

## 9. Warning notice

- Do not operate the Symphonie Aqua Lifter if damage to the device is discovered.
- Check that the lifter is in faultless condition before each use.
- Ship the Symphonie Aqua Lifter only in the original packaging. Liability will not be assumed for damages in transit.
- Modifications or changes to the Symphonie Aqua Lifter result in immediate termination of the guarantee/warranty.
- Keep the lifter out of the reach of children – there is a risk of injury!
- When in operation, the hydraulic fluid of the lever mechanism is under pressure.

- Therefore, the lever mechanism pump may not be opened!
- The fluid should not come into contact with the skin.
- To avoid injury, appropriate personal protective equipment should be used (e. g. safety gloves, protective glasses, as well as skin protection and skin care products)
- Hydraulic fluid is a pollutant. •
  - Avoid spillage. •
  - Spilled hydraulic fluid should be immediately removed with an oil absorbent and disposed of properly.
  - Old hydraulic fluid should be disposed according to regulations.

### Notice:

**If you are unable to take care of the problem yourself, or in case of an intended return, please contact our service in advance:**

### Contact information

Romedis GmbH  
Georg-Wiesböck-Ring 5a  
83115 Neubeuern Germany  
Phone +49 8035 96787896  
Fax +49 8035 96 78 78 12  
[www.romedis.com](http://www.romedis.com)  
info@romedis.com

## **10. Assembly**

### **Warning:**

- Prior to operating, and especially before first usage, check that the entire lifter is in faultless condition.
- There is a possible risk of injury due to sharp edges on the lifter. Please take care when unpacking the lifter.

The Symphonie Aqua Lifter is already completely assembled upon delivery. You only need to remove it from the packaging and choose an appropriate location for it in your workshop.

## **11. Usage in combination with the Symphonie Aqua System**

Once you have chosen an appropriate location in your orthopaedic workshop, you can place the Symphonie Aqua VC TF System onto the lifter platform.

The Symphonie Aqua System can be completely assembled in advance. (For assembly of the Symphonie Aqua System, please see the respective operating instructions).

The Symphonie Aqua Lifter offers you two different functions.

### Function 1:

You can set the desired tilt of the lifter platform by using the small, black handwheel.

### Function 2:

You can vary the height of the platform, as desired.

This works as follows:

Remove the metal rod from the left holding pocket. Connect the metal rod to the intended recess on the pump, on the right side of the lifter.

There is a small lever, directly on the pump.

Turn this lever to the left. Now you can begin to pump the platform upward, with the help of the metal rod as an extension. The small lever remains on the left side during the entire casting process!

When the desired height is reached, you can begin with the casting process.

After completion of the casting process, you can lower the Symphonie Aqua VC TF cylinder. For this purpose, place the small lever again on the right-hand side. The platform will slide down by itself. Please pay attention that no further load takes effect on the platform during its lowering. Otherwise, this could lead to a pump defect.

## **12. Guarantee**

We grant a legal warranty of 24 months for the Symphonie Aqua Lifter. During this period, the guarantee covers material defects, premature wear and manufacturing defects.

**Any modification of the lifter results in immediate termination of the guarantee.**

Regarding the product guarantee, Romedis GmbH reserves the right:

- to repair the damaged product at its own discretion
- to provide substitute parts, free of charge
- to provide a credit voucher for the net sales price

For further information, please contact the customer service at Romedis GmbH.  
([www.romedis.de](http://www.romedis.de))